

Compression Systems for Decongestion

SoftCompress BandageAid
Adjustable compression systems
Targeted oedema treatment plan



Freedom in Motion



Compression systems for decongestion

Self-bandaging

The patient is actively involved in the treatment of oedemas	04
Education and training for the patient	05
Innovative aids for supplementary self-bandaging	05

SoftCompress

Compression bandages	07
Bandages	08
LymphAid and BandageAid	10
Compression roll and compression sheet	13
Additional products	14

Adjustable Compression Wraps 19

ACS Light

Wraps for the lower extremities	21
Wraps for the upper extremities	24
Additional products	26

Compression Wrap

Wraps for the lower extremities	29
Wraps for the upper extremities	32
Additional products	34
Additional information and contraindications	37

Additional products

Skin care / Stocking gloves	38
Laundering care / Adhesive lotion	39
Compression pads	40
Positioning and supporting aid for arm and hand / Protection for bandages	41

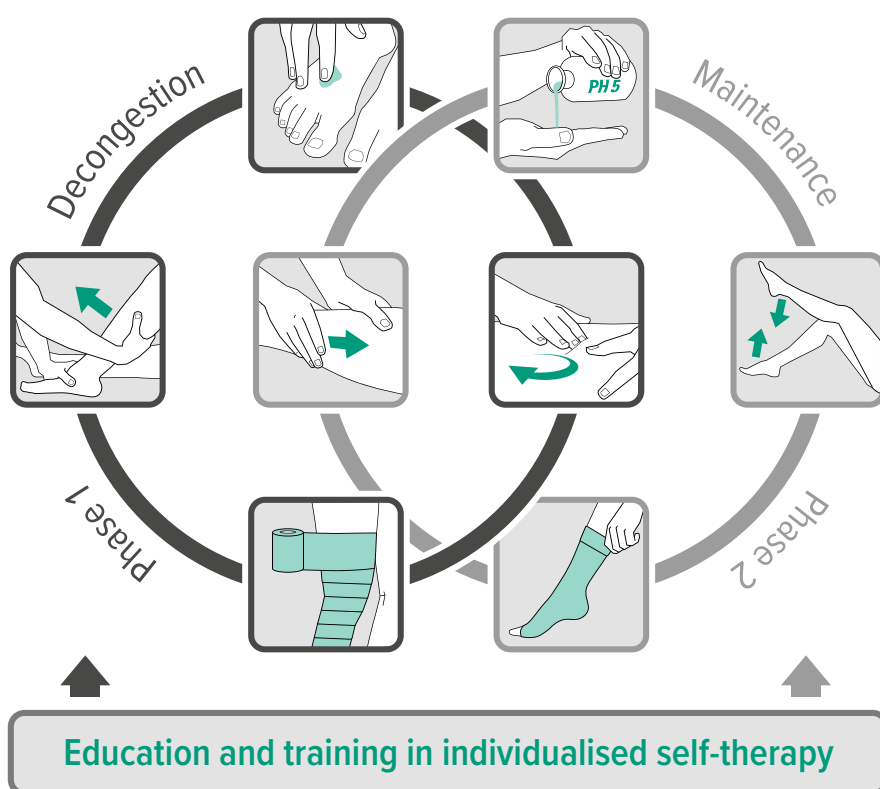
Important information

Dimension sheets as a copy template	42
Checklist / Order form	44

Self-bandaging

The patient is actively involved in the treatment of oedemas.

The standard treatment of a lymphoedema is complex manual decongestive therapy (CDT) which, in accordance with the medical guideline “Diagnosis and treatment of lymphoedema” republished on 23 May 2017, has been complemented by a new component. The active involvement of the patient is new: Education and training for individual self-treatment was included in CDT as the fifth element. All complex manual decongestive therapy measures – manual lymph drainage (MLD), compression therapy, decongesting movement and breath therapy, skin care and skin rehabilitation – should be co-ordinated and used in combination with each other. They can only guarantee long-term treatment success together.



Complex manual decongestive therapy is a 2-phase treatment: Phase I (decongestion phase) is aimed at the mobilisation of the oedema. In phase II (maintenance phase), the therapeutic effect is preserved and optimised. The patient's co-operation and participation are important during every phase and each of the CDT measures.

The aim is to alleviate patients' discomfort, improve their quality of life, help them to actively take part in society and work and to prevent complications. The combination of complex manual decongestive therapy and self-management training ensures the long-term success of the treatment. The effective collaboration between doctor, lymphatic therapist, specialist retailer and patient is key here.

Phase 1: Decongestion phase

In phase I of this treatment, all components of complex manual decongestive therapy should be used 1 – 2 times daily. The education, training and involvement of the patient in his/her treatment is of particular importance in this phase because the measures are time-consuming and require active patient participation.

Phase 2: Maintenance phase

In phase II, the components of complex manual decongestive therapy are applied according to diagnosis and complaints. Phase I may have to be repeated, depending on the progression of the illness – in particular for concomitant conditions. The patient should also be actively involved in the second phase of treatment.

Education and training for the patient

It is key for the success of CDT that patients are educated about their illness and provided with information. Written information is useful at the beginning of treatment. Juzo offers easy-to-understand, free brochures and online information for patients and relatives.

Once patients have understood and accepted their condition, it is easier to speak to them about options. All patients can improve their condition and alleviate discomfort by their own actions. In some cases, psychological guidance or visiting a self-help group can be useful.

As part of patient training, specific questions about the illness and the individual treatment components can be answered. An important aspect is learning to apply lymphological compression bandages, i.e. self bandaging. In addition to compression therapy, which the patient can support actively in this way, there are other options for self-treatment. This includes, for example, appropriate skin care, breathing techniques and decongestion exercises. The patient can learn all these during training.

Self-treatment is a supplementary measure alongside the CDT components defined in the medical guideline. It is not a substitution for manual lymph drainage with compression treatment carried out by therapists or the application of compression stockings by medical retail specialists. All measures should be taken together so that the long-term success of the oedema therapy can be guaranteed and the patient's quality of life can be improved.

Innovative aids for supplementary self-bandaging

Juzo offers numerous products for self-treatment which contribute to an improved quality of life, thanks to their quick and easy application and effectiveness.

The ACS Light and Compression Wrap products as well as Soft-Compress bandaging aids are particularly suitable for patients to put on themselves during the decongestion phase or for further use during the maintenance phase. These products are practical to use and are suitable for every phase of CDT to support treatment success.

Adjustable compression wraps (ACS Light and Compression Wrap) can be adapted to the individual patient using touch fasteners. These products are easy to use and can be put on legs and arms in next to no time. SoftCompress can significantly increase the success of decongestive therapy. Clever product details ensure ideal pressure transfer, preventing excessive shear and friction forces in sensitive areas of the body, for example.

These innovative products can be applied in a targeted and efficient manner depending on the patient's diagnosis and physical situation. Many of these are also available tailor-made with a customised fit. A variety of supplementary products specifically adapted to compression treatment, such as various donning and doffing aids or Lymph Lotion for daily skin care facilitate effective application.

In order to carry out successful oedema treatment long-term, product solutions that are characterised by a quick and easy application are required.





SoftCompress

Compression bandages

When compression bandaging is used, padding underneath prevents pressure points and skin damage. It can also improve quality of life¹. In addition, the SoftCompress cushioning material can significantly increase the success of decongestive therapy. This is thanks to the vertical foam channels that form a unique surface structure to support lymphatic drainage. Bandaging aids and additional products feature a particularly skin-friendly cotton surface, as opposed to frequently used synthetic materials. This provides the patient with pleasant wearing comfort. In addition, SoftCompress bandaging aids make it easy for patients to use the bandages themselves, therefore ensuring the long-term success of treatment.

Features

- + core made of high-quality foam
 - optimum transfer of pressure
 - perfect pressure distribution
- + vertically-stitched foam channels
 - increased lymph drainage
 - effective for counteracting fibrosis
- + exceptionally skin-friendly, thanks to the cotton surface
 - breathable
 - smooth
- + constriction in the bends of the joints is prevented
- + protects against bandage edges pinching
- + saves time when bandaging
- + easier self-bandaging
- + washable up to 60 °C
- + suitable for tumble-drying at a low temperature
- + disinfection and treatment possible – therefore saves money²
- + **SoftCompress bandages can be worn under shortstretch bandages or in combination with adjustable compression wraps**

Indications

- lymphoedema
- lipoedema
- phleboedema
- post-traumatic oedema
- postoperative oedema
- fibrosis

Material composition

- Top cover: 100 % cotton
- Foam: 100 % polyurethane

Contraindications

See page 37

Colour



White



With cotton



Washable
60 °C



Suitable for
tumble-drying



Foam channels

Vertically-stitched foam channels promote lymphatic drainage and help to loosen fibrosis.



Effectiveness

Visible effects of SoftCompress bandages in cases of secondary leg lymphoedema.

¹Protz et al. "Kompressionsbandagierungen mit und ohne Unterpolsterung im Vergleich" (Comparing compression bandaging with and without cushioning). Vasomed 2019; 31(3) 132-133.

²The products have been successfully tested in accordance with DIN ISO TS 15883-5, SOD 19-002, EN ISO 17664, SOD 19-01 and the Robert Koch Institute "Hygiene Requirements for the Reprocessing of Medical Devices" guideline.

Bandages

In phase 1 of CDT, the comfortable SoftCompress bandages can be worn under short-stretch bandages or, in phase 2, under medical compression stockings or ACS Light. They provide additional compression, cushioning and an ideal pressure distribution. The bandages enable bandaging to be carried out more quickly and effectively. They also make it easier for patients to apply the bandages themselves. They are available in uni size, standard size or custom-made. Custom-made bandages can also be used for treatment at night.

For CDT phase 1



**SoftCompress
Bandage Lower Leg**

in uni size

Art. 6831



**SoftCompress
Bandage Lower Leg and Thigh**

in uni size

Art. 6821



**SoftCompress
Bandage Arm**

in uni size

Art. 6823

Uni size

Uni size bandages are open on the back and are secured in place with the help of touch fasteners or an elastic fabric sleeve. This enables the bandaging aid to accommodate large fluctuations in circumference during therapy.

Standard and uni sizes can be trimmed at the markings provided.

Caution: Before trimming between the cross-seams, the product has to be washed once and stretched lengthwise while it's still wet, before drying.



For CDT phase 2



**SoftCompress
Bandage Lower Leg**

in standard sizes or custom-made

Art. 6830



**SoftCompress
Bandage Lower Leg and Thigh**

custom-made

Art. 6820









**SoftCompress
Bandage Arm**

in standard sizes or custom-made

Art. 6822

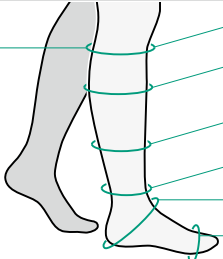
Product range

Art.	Model	Standard sizes	Custom-made	Uni size
	6831 Bandage lower leg (for CDT phase 1)			
	Incl. 2 malleo pads	—	•	•
	6830 Bandage lower leg (for CDT phase 2)			
		1 – 5	•	—
	6821 Bandage lower leg and thigh (for CDT phase 1)			
	Incl. bandage lower leg, uni size 2 malleo pads 1 knee pad 1 compression roll (10 cm x 4 m) 1 compression roll (12 cm x 4 m))	—	—	•
	6820 Bandage lower leg and thigh (for CDT phase 2)			
	Incl. 2 malleo pads 1 knee pad	—	•	—
	6823 Bandage arm (for CDT phase 1)			
	Incl. 1 hand pad 1 elbow pad	—	—	•
	6822 Bandage arm (for CDT phase 2)			
	Incl. 1 hand pad 1 elbow pad	1 – 6	•	—

CDT= Complex decongestive therapy

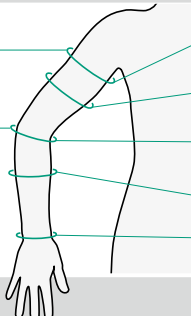
Sizing charts

Juzo SoftCompress Bandage Lower Leg

Art. 6830 Length in cm		Standard sizes - Circumferences in cm				
		1 / XS	2 / S	3 / M	4 / L	5 / XL
	PD 38.0 – 46.0	cD 27.0 – 29.5	29.5 – 32.5	32.5 – 35.5	35.5 – 39.0	38.5 – 42.0
		cC 29.0 – 31.5	31.5 – 34.5	34.5 – 37.5	37.5 – 41.0	40.5 – 44.5
		cB' 23.5 – 25.5	25.5 – 28.0	27.5 – 30.0	30.0 – 33.0	32.5 – 35.5
		cB 18.5 – 20.0	20.0 – 22.0	22.0 – 24.0	24.0 – 26.0	26.0 – 28.5
		cY 28.0 – 30.5	29.0 – 31.5	31.0 – 34.0	33.0 – 36.0	35.0 – 38.5
		cA 18.5 – 20.0	20.0 – 22.0	22.0 – 24.0	24.0 – 26.0	26.0 – 28.5

Important:
Standard sizes
are determined by
skin dimensions. For
custom-made
products, please
measure the item
"on tension".

Juzo SoftCompress Bandage Arm

Art. 6822 Lengths in cm		Standard sizes - Circumferences in cm					
		1 / XS	2 / S	3 / M	4 / L	5 / XL	6 / XXL
	PG 46.0 – 51.0	cG 30.0 – 33.0	32.0 – 35.0	34.0 – 37.0	36.0 – 39.5	38.0 – 41.5	40.0 – 44.0
		cF 28.0 – 30.5	30.0 – 33.0	32.0 – 35.0	34.0 – 37.0	36.0 – 39.5	38.0 – 41.5
		cE 25.0 – 27.5	27.0 – 29.5	29.0 – 31.5	30.5 – 33.5	32.0 – 35.0	33.5 – 36.5
		cD 23.0 – 25.0	24.5 – 27.0	26.0 – 28.5	27.5 – 30.0	29.0 – 31.5	30.5 – 33.5
	PE 21.0 – 25.0	cC 14.5 – 16.0	16.0 – 17.5	17.5 – 19.0	19.0 – 20.5	20.5 – 22.0	22.0 – 24.0

LymphAid and BandageAid

SoftCompress LymphAid and BandageAid offer a complete solution for lymphological compression bandaging. Unlike conventional sets, they also include the innovative SoftCompress materials which can significantly improve treatment outcomes. LymphAid comes with a ready-to-use bandage and is therefore also suitable for self-bandaging. The more economical BandageAid contains SoftCompress Compression Rolls.

LymphAid



**SoftCompress
LymphAid Leg, small**
Art. 6800 KL

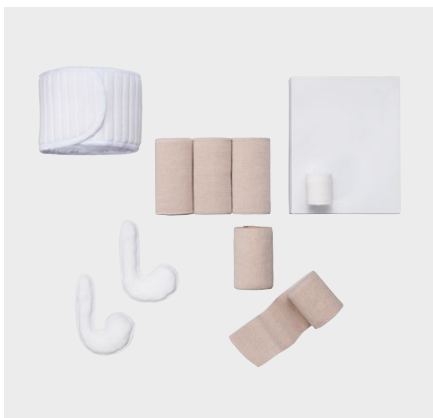


**SoftCompress
LymphAid Leg, large**
Art. 6800 GR



**SoftCompress
LymphAid Arm**
Art. 6801

BandageAid



**SoftCompress
BandageAid Leg, small**
Art. 6810 KL









**SoftCompress
BandageAid Leg, large**
Art. 6810 GR



**SoftCompress
BandageAid Arm**
Art. 6811

Product range

Art.	Model	Uni size
	6800 KL LymphAid Leg, small (for CDT phase 1)	
	1 lower leg bandage, uni size 20 Rapedelast V – gauze bandages 4 cm x 4 m 2 short-stretch bandages 6 cm x 5 m 2 short-stretch bandages 8 cm x 5 m 2 short-stretch bandages 10 cm x 5 m 4 short-stretch bandages 12 cm x 5 m 2 malleo pads	.
	6800 GR LymphAid Leg, large (for CDT phase 1)	
	1 lower leg bandage, uni size 1 compression roll 10 cm x 4 m 1 compression roll 12 cm x 4 m 20 Rapedelast V – gauze bandages 4 cm x 4 m 2 short-stretch bandages 6 cm x 5 m 2 short-stretch bandages 8 cm x 5 m 4 short-stretch bandages 10 cm x 5 m 6 short-stretch bandages 12 cm x 5 m 2 malleo pads 1 knee pad	.
	6801 LymphAid Arm (for CDT phase 1)	
	1 arm bandaging aid, uni size 20 Rapedelast V – gauze bandages 6 cm x 4 m 2 short-stretch bandages 6 cm x 5 m 2 short-stretch bandages 10 cm x 5 m 3 short-stretch bandages 12 cm x 5 m 1 hand pad 1 elbow pad	.
	6810 KL BandageAid Leg, small (for CDT phase 1)	
	1 compression roll 10 cm x 4 m 20 Rapedelast V – gauze bandages 4 cm x 4 m 1 short-stretch bandage 6 cm x 5 m 1 short-stretch bandage 8 cm x 5 m 3 short-stretch bandages 10 cm x 5 m 2 malleo pads	.
	6810 GR BandageAid Leg, large (for CDT phase 1)	
	1 compression roll 10 cm x 4 m 2 compression rolls 12 cm x 4 m 20 Rapedelast V – gauze bandages 4 cm x 4 m 1 short-stretch bandage 6 cm x 5 m 1 short-stretch bandage 8 cm x 5 m 3 short-stretch bandages 10 cm x 5 m 4 short-stretch bandages 12 cm x 5 m 2 malleo pads 1 knee pad	.
	6811 BandageAid Arm (for CDT phase 1)	
	1 compression roll 10 cm x 4 m 20 Rapedelast V – gauze bandages 6 cm x 4 m 1 short-stretch bandage 6 cm x 5 m 1 short-stretch bandage 8 cm x 5 m 5 short-stretch bandages 10 cm x 5 m 1 hand pad 1 elbow pad	.



Application videos:

LymphAid, lower leg



LymphAid, leg



BandageAid, arm and hand



LymphAid, lower leg



BandageAid, leg



BandageAid, arm and hand



CDT= Complex decongestive therapy

The gauze bandages and short-stretch bandages used in LymphAid and BandageAid are quality products manufactured by Karl Otto Braun GmbH & Co. KG (KOB). They are also available separately.



Compression roll and compression sheet

SoftCompress bandaging materials are used for lymphological and phlebological compression bandaging. Thanks to their soft surface, they are very smooth, skin-friendly and breathable.



SoftCompress Compression Roll

Art. 6835

The thin and smooth SoftCompress Compression Rolls exert compression when applied correctly. The compression rolls evenly distribute the pressure of the short-stretch bandage applied on top, thus preventing constriction.



SoftCompress Compression Sheet

Art. 6836

The SoftCompress Compression sheets can be cut to size and can therefore be used in a number of different body regions. We recommend stitching down the edges.

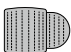
Product range

Art.	Model	Standard sizes
6835	Compression Roll (for CDT phase 1)	
	5 mm, 8 cm x 4 m 5 mm, 10 cm x 4 m 5 mm, 12 cm x 4 m 5 mm, 16 cm x 4 m	• • • •
6836	Compression Sheet (for CDT phase 1)	
	65 x 65 cm 125 x 100 cm	• •


CDT= Complex decongestive therapy

Sizing charts

Juzo SoftCompress Compression Roll

Art. 6835	Standard sizes	Measurements in cm (length x breadth)
Thickness in mm: 5.0 	1	400.0 x 8.0
	2	400.0 x 10.0
	3	400.0 x 12.0
	4	400.0 x 16.0

Juzo SoftCompress Compression Sheet

Art. 6836	Standard sizes	Measurements in cm (breadth x height)
	1	65.0 x 65.0
	2	125.0 x 100.0

Additional products

SoftCompress

SoftCompress pads can be worn underneath compression bandages or compression garments. They are especially suited to providing a targeted increase in pressure in localised oedemas.



SoftCompress Breast Pad

Art. 6837

The SoftCompress Breast Pad serves as a comfortable pad for a compression bra or thorax compression vest and can be used for breast or chest wall oedemas, for example.



SoftCompress Breast Bandage

Art. 6838

The SoftCompress Breast Bandage combines the positive characteristics of the SoftCompress material with those of a compression bra. The bandage has an adjustable chest circumference. The breast bandage can be used for breast or chest wall oedema, for example.



SoftCompress Genital Pad for women

Art. 6839

The SoftCompress Genital Pad for women is inserted into compression tights. It provides a targeted increase in pressure, cushioning and an ideal pressure distribution when treating genital oedemas.



SoftCompress Genital Pad for men

Art. 6840

The SoftCompress Genital Pad for men (reversible) is inserted into compression tights. The wide channel structure helps to increase pressure and lymph drainage, while the narrow channels are useful for reducing hardening caused by fibrosis. It also provides a targeted increase in pressure and an ideal pressure distribution when treating genital oedemas.



SoftCompress Knee Pad

Art. 6845

This SoftCompress pad provides cushioning for the back of the knee joint.



SoftCompress Elbow Pad

Art. 6842

This SoftCompress pad provides cushioning where needed. It is especially useful for the crook of the elbow.



SoftCompress Malleo Pad

Art. 6843

SoftCompress Malleo Pads cushion the Bisgaard (perimalleolar) region of the ankle. This ensures a targeted increase in pressure and an ideal pressure distribution. The pads come in pairs.



SoftCompress Hand Pad

Art. 6844

SoftCompress Hand Pads are specially designed for the back of the hand and feature two finger openings. They ensure a targeted increase in pressure and an ideal pressure distribution.



SoftCompress Short-stretch Bandage

Art. 6846

The short-stretch bandages are made from 100 % cotton, creating compression bandaging with low resting pressure and high working pressure. They can be washed up to 95 °C and sterilised with steam (121 °C).



SoftCompress Rapidelast V gauze bandages

Art. 6847/6848

The Rapidelast V gauze bandages are made from 60 % polyamide and 40 % viscose. They are used to secure compression and cushioning material – especially on sensitive skin or joints – without restricting the patient's freedom of movement. Bandages with Rapidelast V are non-slip. The high elasticity of the fabric makes application quick and easy.



SoftCompress Tubular Bandages

Art. 6849

The tubular bandage is made from 100 % cotton. It is particularly stretchy and gentle on the skin. It protects the skin without constricting it and can also be used to secure pads.


Product range

Art.	Model	Standard sizes	Custom-made
6837	Breast pad (for CDT phases 1 and 2)		
	Cup A – B Cup C – D Cup E – F	1 – 3	•
6838	Breast bandage (for CDT phases 1 and 2)		
	Cup A Cup B Cup C Cup D Cup E Cup F	1 – 6	•
6839	Genital pad for women (for CDT phases 1 and 2)		
	20 x 20 cm 30 x 28 cm 40 x 32 cm	1 – 3	•
6840	Genital pad for men (for CDT phases 1 and 2)		
	20 x 28 cm 23 x 30 cm 26 x 32 cm	1 – 3	•
6842	Elbow pad (for CDT phases 1 and 2)	Uni size	
	10 x 13 cm (custom-made max. 13 x 16 cm)	•	•
6843	Malleo pads (for CDT phases 1 and 2)	Uni size	
	6.5 x 9 cm delivered in pairs	•	•
6844	Hand pad (for CDT phases 1 and 2)	Uni size	
	8.5 x 24 cm	•	•
6845	Knee pad (for CDT phases 1 and 2)	Uni size	
	13 x 17 cm (custom-made max. 19 x 23 cm)	•	•

Art.	Model	Each	Box
6846	Short-stretch bandage (for CDT phase 1)		10 items
	6 cm x 5 m 8 cm x 5 m 10 cm x 5 m 12 cm x 5 m	• • • •	• • • •
	Rapidelast V gauze bandage (for CDT phase 1)		20 items
6847 6848	6 cm x 4 m (for the fingers) 4 cm x 4 m (for the toes)	– –	• •
6849	Tubular bandage (for CDT phase 1)		
	6 cm x 20 m 8 cm x 20 m 10 cm x 20 m	• • •	– – –

Sizing charts

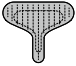
Juzo SoftCompress Breast Pad

Art. 6837	Standard sizes	Cup
	1	A – B
	2	C – D
	3	E – F


Juzo SoftCompress Breast Bandage

Art. 6838	Standard sizes	Cup	Breast circumference in cm	Underbreast circumference in cm
	1	A	90.0 – 106.0	84.0 – 100.0
	2	B	90.0 – 106.0	84.0 – 100.0
	3	C	95.0 – 112.0	85.0 – 102.0
	4	D	95.0 – 112.0	85.0 – 102.0
	5	E	100.0 – 116.0	88.0 – 104.0
	6	F	100.0 – 116.0	88.0 – 104.0

Juzo SoftCompress Genital Pad Women

Art. 6839	Standard sizes	Measurements in cm (breadth x height)
	1	20.0 x 20.0
	2	30.0 x 28.0
	3	40.0 x 32.0

Juzo SoftCompress Genital Pad Men

Art. 6840	Standard sizes	Measurements in cm (breadth x height)
	1	20.0 x 28.0
	2	23.0 x 30.0
	3	26.0 x 32.0



Adjustable compression systems

ACS Light and Compression Wrap

Adjustable compression systems (ACS) can be used in the decongestive treatment phase as an efficient alternative to conventional compression rolls. These innovative products enable health care professionals to save time and money¹ while effectively promoting treatment success in decongestive therapy². In addition, the treatment success of manual lymphatic drainage can be supported by the patient self-therapy. Compression systems can be worn during the day and at night and they can be adapted to the individual patient – even if circumferences change during treatment. Easy handling of the touch fasteners means that patients can take care of the adjustment themselves to ensure the compressive effect is perfectly maintained with increased decongestion³. Open wounds must first be covered with wound dressing before the compression system can be put on. Wearing a liner stocking is recommended for the lower extremity segments. Alternatively, the combination with SoftCompress bandages (p. 8) is an ideal treatment option that additionally boosts treatment success.

Features

- + short-stretch
- + material is especially durable
- + can be worn day and night
- + adaptable whilst wearing – for ideal compression
- + easy to apply yourself – even with limited mobility or strength
- + easy to put on by the patient
- + **Adjustable compression systems can also be combined with SoftCompress products**

Indications

- lymphoedema
- lipoedema
- phleboedema
- Ulcus cruris venosum
- post-traumatic oedema
- postoperative oedema
- deep vein thrombosis
- post-thrombotic syndrome
- chronic venous insufficiency
- primary and secondary varicose veins
- immobility oedema
- work-related oedema (work standing up / sitting down)

Contraindications

See page 37

¹ Dissemmond, S. et al.: "Praxisrelevante Aspekte der Kompressionstherapie für Patienten mit chronischem Ulcus cruris unter besonderer Berücksichtigung der Wirtschaftlichkeit" (Practical aspects of compression therapy for patients suffering from chronic venous leg ulcers with particular consideration of economy). WUNDmanagement 2018; 12(2) 77-84.

² Damstra, R. & Partsch, H.: Randomized controlled trial comparing the effectiveness of adjustable compression Velcro wraps versus inelastic multicomponent compression bandages in the initial treatment of leg lymphedema. Journal of Vascular Surgery 2013; 1(1) 13-19.

³ Mosti, G. & Partsch, H.: "Druckmessungen unter Klettverschluss-Kompression" (Pressure measurement under touch fastener compression). Vasomed 2017; 29(5) 212-216.

ACS Light

Adjustable compression systems

Owing to its short-stretch properties, the ACS Light exerts a low resting pressure and a high working pressure. The individual segments are easy to put on, thanks to the integrated lining (fixing aid). Adhesive borders on the top edge of a segment guarantee a secure hold at all times. Last but not least, the comfortable material increases patient compliance and supports self-therapy.

Features

- + compression levels up to 40 mmHg
- + adaptable whilst wearing – for ideal compression
- + perfect hold, thanks to adhesive borders on the inside
- + easy to put on by the patient, thanks to the sewn-in fixing aid
- + various accessories available
- + disinfection and treatment possible
- + **ACS Light segments can also be combined with SoftCompress products**

Material composition

85 % polyamide, 15 % elastane

Contraindications

See page 37

Colours



Beige

Black



High working pressure



Standard sizes



Velcro fasteners

Velcro fasteners make it easy to apply the product and to adjust it in case of fluctuations in body circumference without taking the product down.



Secure hold

Thanks to the incorporated lining (fixing aid), the segments can be put on in next to no time and provide a perfect fit. The sewn-in silicone border prevents slipping.



Wearing comfort

A separate pad included with the arm and hand wrap offers additional comfort.



Anatomical fit

The compression wraps are perfectly adapted to individual body contours. The wraps for the lower leg, thigh and arm are available for both the left and right side of the body.

* The products have been successfully tested in accordance with DIN ISO TS 15883-5, SOD 19-002, EN ISO 17664, SOD 19-01 and the Robert Koch Institute "Hygiene Requirements for the Reprocessing of Medical Devices" guideline.

Lower extremity

ACS Light variable compression system for the lower extremities are compression segments with short-stretch characteristics, intended to reduce oedema in the foot, lower leg or the entire leg. Equipped with adjustable straps, they can be adapted to fit various circumferences, even if those change during treatment. These are highly suited to patients' self-bandaging.

Segments for the lower leg and foot can be combined. If required, wraps for the knee or the thigh can be added to treat the entire leg. Liner, available separately, is recommended as liner stockings.



ACS Light Thigh Wrap

- + with sewn-in silicone border
- + with incorporated lining (fixing aid)
- + Includes separate extender (extension)
- + available with extender in left and right versions



ACS Light Knee Wrap

- + with sewn-in silicone border
- + with incorporated lining (fixing aid)
- + can be worn both left and right
- + with knee comfort hole



ACS Light Calf Wrap

- + with sewn-in silicone border
- + with incorporated lining (fixing aid)
- + available in left and right versions



ACS Light Foot Wrap





- + can be worn both left and right
- + with incorporated malleo pads
- + Includes separate extender to increase pressure



Product range

ACS Light for the lower extremities

Adjustable compression systems

Art.	Model	Standard sizes			
	Length				
J701LI	Foot wrap	Beige		Black	
	Normal	1 – 5		1 – 5	
	Long				
J702LI	Calf wrap	Beige/left	Beige/right	Black/left	Black/right
	Normal	1 – 5	1 – 5	1 – 5	1 – 5
	Long				
J703LI	Knee wrap	Beige		Black	
	Normal	1 – 5		1 – 5	
J704LI	Thigh wrap	Beige/left	Beige/right	Black/left	Black/right
	Normal	1 – 5	1 – 5	1 – 5	1 – 5
	Long				



Application video:
ACS Light, leg




Sizing charts

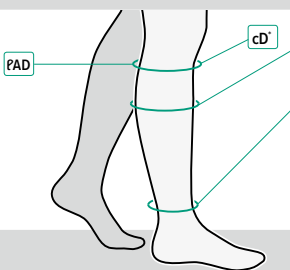
ACS Light for the lower extremities

Adjustable compression systems


Juzo ACS Light Foot Wrap

Art. J701LI	Circumference c		Standard sizes - Measurements in cm				
	Length P		1 / XS	2 / S	3 / M	4 / L	5 / XL
	cA	over forefoot	22.0 – 24.0	25.0 – 27.0	28.0 – 30.0	31.0 – 34.0	35.0 – 40.0
	PA	normal	16.5	18.0	19.5	21.0	23.0
	PA	long	18.5	20.0	21.5	23.0	25.0

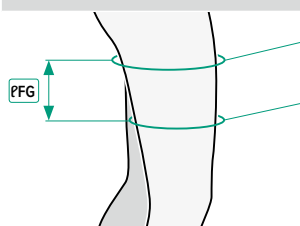
Juzo ACS Light Calf Wrap

Art. J702LI	Circumference c Length P		Standard sizes - Measurements in cm				
			1 / XS	2 / S	3 / M	4 / L	5 / XL
	cC	largest point of calf	35.0 – 45.0	40.0 – 52.0	48.0 – 60.0	55.0 – 65.0	60.0 – 70.0
	cB	ankle	20.0 – 27.0	25.0 – 32.0	30.0 – 38.0	37.0 – 44.0	45.0 – 52.0
	PAD	normal	37.0	37.0	42.0	42.0	44.0
	PAD	long	42.0	42.0	47.0	47.0	49.0
		* below patella					

Juzo ACS Light Knee Wrap

Art. J703LI	Circumference c	Standard sizes - Measurements in cm				
		1 / XS	2 / S	3 / M	4 / L	5 / XL
	cF 10 cm above patella	40.0 – 55.0	50.0 – 65.0	60.0 – 75.0	70.0 – 85.0	85.0 – 100.0
	cD below patella	35.0 – 45.0	40.0 – 52.0	48.0 – 60.0	55.0 – 65.0	60.0 – 70.0

Juzo ACS Light Thigh Wrap

Art. J704LI	Circumference c		Standard sizes - Measurements in cm				
	Length P		1 / XS	2 / S	3 / M	4 / L	5 / XL
	cG	largest point of thigh	50.0 – 65.0	60.0 – 75.0	70.0 – 80.0	75.0 – 90.0	85.0 – 100.0
	cF	10 cm above patella	45.0 – 52.0	53.0 – 59.0	60.0 – 66.0	67.0 – 73.0	74.0 – 80.0
	PFG	normal	23.0	23.0	23.0	23.0	23.0
	PFG	long	29.0	29.0	29.0	29.0	29.0

Upper extremity



ACS Light variable compression system for the upper extremities are compression segments with short-stretch characteristics, intended to reduce oedema in the arm and hand. Equipped with adjustable straps, they can be adapted to fit various circumferences, even if those change during treatment. These are highly suited to patients' self-bandaging. Segments for the arm and hand can be combined. The hand wrap is particularly suited for treating oedema in the area of the back of the hand.



Product range

ACS Light for the upper extremities

Adjustable compression systems

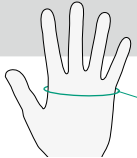
Art.	Model Length	Standard sizes	
J705LI	Hand wrap	Beige	Black
	Normal	1 – 4	1 – 4
J706LI	Arm wrap	Left	Right
	Normal	1 – 5	1 – 5
	Long		

Sizing charts

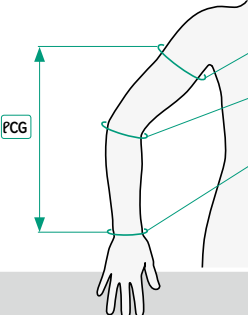
ACS Light for the upper extremities

Adjustable compression systems

Juzo ACS Light Hand Wrap

Art. J705LI	Circumference c 	Standard sizes - Measurements in cm			
		1 / XS	2 / S	3 / M	4 / L
	cA over knuckles	< 18.0	18.0 – 20.0	20.0 – 22.0	> 22.0

Juzo ACS Light Arm Wrap

Art. J706LI	Circumference c Length l	Standard sizes - Measurements in cm				
		1 / XS	2 / S	3 / M	4 / L	5 / XL
	cG upper arm	25.0 – 28.0	29.0 – 33.0	34.0 – 39.0	40.0 – 45.0	46.0 – 50.0
	cE elbow	23.0 – 26.0	26.0 – 30.0	29.0 – 34.0	33.0 – 39.0	38.0 – 42.0
	cC wrist	14.0 – 16.0	17.0 – 19.0	20.0 – 22.0	23.0 – 25.0	24.0 – 27.0
	PCG normal	42.0 – 47.0	42.0 – 47.0	42.0 – 47.0	42.0 – 47.0	42.0 – 47.0
	PCG long	46.0 – 50.0	46.0 – 50.0	46.0 – 50.0	46.0 – 50.0	46.0 – 50.0



Application video:
ACS Light, arm



Additional products

ACS Light

Adjustable compression systems

Several additional products are available for ACS Light to provide secure fastening, extra wearing comfort and care of wider circumferences. These are easy to use and also highly suited to patients' self-bandaging.



ACS Light Hip Band

Art. J711

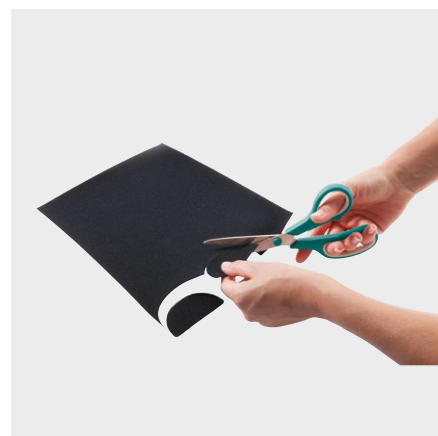
The ACS Light Hip Band can be used for fixing the thigh wrap, if required.



ACS Light Midline Band

Art. J712

The ACS Light Midline Band has been specially developed for the compression of the torso (e.g. thorax, pelvis, abdomen and/or genital area). It can be cut to size and is easy to put on thanks to the general-purpose touch fasteners.



ACS Light Touch Fastener Sheet

Art. J713

With the ACS Light Velcro Fastener Sheet, additional touch fasteners can be individually cut to size.



ACS Light Pads

Art. J714

ACS Light Pads can be used to provide extra wearing comfort with additional cushioning and/or to provide increased pressure in specific areas.



ACS Light Anti-slip Band

Art. J715

The ACS Light Anti-slip Band offers an additional fastening option and can be shortened as required. It can be worn under the midline band and hip band as well as under the thigh wrap to provide secure fastening.



ACS Light Extender

Art. J716

The ACS Light Extender provides increased compression for the extremities (e.g. the ankle, back of the hand or instep).

Product range


Art.	Model		
J711	Hip band	Uni size · Beige	Uni size · Black
	Hip section with sewn-on micro touch fastener separate waist band including 1 loose touch fastener Colours: Beige, Black	•	•
J712	Midline band	Standard size · Beige	Standard size · Black
	Additional band including 6 loose touch fasteners Colours: Beige, Black	1 – 2	1 – 2
J713	Touch fastener sheet	Uni size · Beige	Uni size · Black
	Can be individually cut to size Colours: Beige, Black	•	•
J714	Pads	Standard size · Grey	
	Set with 2 thicknesses Colours: Grey	1 – 4	
J715	Anti-slip band	Uni size · Beige	Uni size · Black
	Band with neoprene coating on one side, 12 x 120 cm including 6 loose touch fasteners Colours: Beige, Black	•	•
J716	Extender	Uni size · Beige	Uni size · Black
	Extension band with sewn-on touch strips including 2 loose touch fasteners Colours: Beige, Black	•	•

Sizing charts

Juzo ACS Light Midline Band

Art. J712	Standard sizes	Measurements in cm (length x width)
	1	140.0 x 12.0
	2	140.0 x 20.0

Juzo ACS Light Pads

Art. J714	Standard sizes	Measurements in cm (length x width)
Thickness in mm: 4.0 and 8.0 	1	9.0 x 9.0
	2	10.0 x 10.0
	3	11.0 x 11.0
	4	12.0 x 12.0

Compression Wrap

Adjustable compression system

Based on their extremely short-stretch properties, Compression Wrap products exert very high working pressure and low resting pressure. The double-sided touch fasteners and helpful gripping zones ensure easy donning and adjustment in the event of fluctuations in body circumference. Special indentations on the tabs ensure complete closure and prevent the formation of “window oedemas.” Special feature: the product is reversible, with black on one side and beige on the other, to suit the individual patient’s preference.

Features

- + variable compression 30 – 60 mmHg
- + can be individually adjusted and cut to size when treating fluctuating leg circumferences
- + special indentations ensure the perfect overlap of the tabs, thus preventing the formation of “window oedemas”
- + gripping zones at the upper and lower end make donning easier
- + the product is reversible, with black on one side and beige on the other, to suit the individual patient’s preference
- + **Compression Wrap segments can also be combined with Soft-Compress products**

Material composition

Outer material: 88 % polyamide, 12 % elastane
Inner material: 100 % neoprene

Contraindications

See page 37

Colours



Beige / Black



very high working pressure



reversible



standard sizes



Special indentations

Thanks to this fixture on the tabs, the formation of “window oedemas” is prevented. The indentations ensure that the individual tabs overlap without gaps.



Helpful gripping zones

Practical zones at the upper and lower end make closing the tabs easier.



Individual colour selection

Thanks to the special material processing, the product is reversible, with black on one side and beige on the other.

Lower extremity

Compression Wrap variable compression system for the lower extremities are compression segments with short-stretch characteristics, intended to reduce oedema in the foot, lower leg or the entire leg. Equipped with adjustable straps, they can be adapted to fit various circumferences, even if those change during treatment. These are highly suited to patients' selfbandaging. Segments for the lower leg and foot can be combined.

NEW If required, wraps for the knee or the thigh can be added to treat the entire leg. Liner, available separately, is recommended as liner stockings.



Compression Wrap Thigh

- + special indentations allow for the perfect overlap
- + gripping zones for easier donning
- + can be worn both left and right
- + the product is reversible, with black on one side and beige on the other

NEW

Compression Wrap Knee

- + special indentations allow for the perfect overlap
- + gripping zones for easier donning
- + can be worn both left and right
- + the product is reversible, with black on one side and beige on the other

NEW

Compression Wrap Calf

- + with incorporated lining (fixing aid)
- + includes liner stocking
- + special indentations allow for the perfect overlap
- + gripping zones for easier donning
- + can be worn both left and right
- + the product is reversible, with black on one side and beige on the other






Compression Wrap Foot

- + special indentations allow for the perfect overlap
- + gripping zones for easier donning
- + can be worn both left and right
- + the product is reversible, with black on one side and beige on the other

Product range

Compression Wrap for lower extremities

Adjustable compression system

	Art. Length	Model	Unit	Standard sizes
	6000LF	Foot wrap	Each	
	Normal	NEW: Size 5		2 – 5
	Long NEW			
	6000LC	Calf wrap	Each	
	Normal			2 – 7
	Long			
	6000LCM	Calf wrap max.	Each	
	Normal			2 – 7
	Long			
	6000LK	Knee wrap	Each	
				2 – 7
	6000LT	Thigh wrap	Each	
	Normal			2 – 6
	6000LTg			
	Long			

Sizing charts

Compression Wrap for lower extremities

Adjustable compression system

Juzo Compression Wrap Foot

Art. 6000 LF/LFL		Standard sizes - Circumferences in cm					
		2 / S	3 / M	4 / L	5 / XL		
	cB	15.0 – 23.0	20.0 – 28.0	25.0 – 33.0	30.0 – 40.0		
	cA**	19.0 – 24.0	24.0 – 29.0	29.0 – 34.0	34.0 – 40.0		
	PA	normal	16	18	19.5	19.5	
		long	19	21	22.5	23	

* 6 cm proximal to the little toe metatarsophalangeal joint

Juzo Compression Wrap Calf

Art. 6000 LC/LLC/LCM/LLCM		Standard sizes - Circumferences in cm					
Lengths in cm		2 / S	3 / M	4 / L	5 / XL	6 / XXL	7 / XXXL
	PD AD	29.0 – 35.0	34.0 – 40.0	37.0 – 44.0	41.0 – 48.0	46.0 – 53.0	50.0 – 60.0
	normal < 40,0	35.0 – 41.0	40.0 – 46.0	44.0 – 51.0	48.0 – 55.0	53.0 – 60.0	60.0 – 70.0
	long > 40,0	18.0 – 21.0	21.0 – 24.0	24.0 – 27.0	27.0 – 31.0	31.0 – 35.0	36.0 – 47.0

Juzo Compression Wrap Knee

Art. 6000 LK		Standard sizes - Circumferences in cm					
		2 / S	3 / M	4 / L	5 / XL	6 / XXL	7 / XXXL
	cE	30.0 – 36.0	36.0 – 42.0	42.0 – 48.0	48.0 – 55.0	55.0 – 62.0	62.0 – 70.0

Juzo Compression Wrap Thigh

Art. 6000 LT/LTg		Standard sizes - Circumferences in cm				
Lengths in cm		2 / S	3 / M	4 / L	5 / XL	6 / XXL
	PFG	43.0 – 70.0	54.0 – 81.0	65.0 – 92.0	75.0 – 102.0	85.0 – 112.0
	normal < 25,0	35.0 – 55.0	48.0 – 70.0	60.0 – 82.0	70.0 – 92.0	80.0 – 102.0
	long > 30,0					



Application videos:
Compression Wrap, lower leg




Compression Wrap, leg



Upper extremity

Compression Wrap variable compression system for the upper extremities are compression segments with short-stretch characteristics, intended to reduce oedema in the arm and hand. Equipped with adjustable straps, they can be adapted to fit various circumferences, even if those change during treatment. These are highly suited to patients' self-bandaging. Segments for the arm and hand can be combined. The hand wrap is particularly suited for treating oedema in the area of the back of the hand.



Compression Wrap Arm

- + with incorporated lining (fixing aid)
- + special indentations allow for the perfect overlap
- + can be worn both left and right
- + both sides of the product can be used; the colour can therefore be chosen individually

NEW

Compression Wrap Hand

- + special indentations allow for the perfect overlap
- + available with extender in left and right versions
- + the side can be chosen based on the preferred colour*

NEW

*Please note:

in order to achieve perfect wearing comfort, the side of the hand wrap (right or left) must be selected according to the colour. If the hand wrap is to be worn with beige on the outside, the desired colour must be chosen during the order. If black is preferred on the outside, the opposite side must be ordered.

Example:

Required: hand wrap, size M, for the left hand:

- Order in **beige**: 6000 UHL, size M, **left**
- Order in **black**: 6000 UHR, size M, **right**





Application video:
Compression Wrap, arm



Product range

Compression Wrap for upper extremities

Adjustable compression system

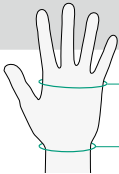
	Art.	Model	Unit	Standard sizes
	Length			
	6000UHL	Hand wrap left	Each	
	Beige: left Black: right			1 – 5
	6000UHR	Hand wrap right	Each	
	Beige: right Black: left			1 – 5
	6000UA	Arm wrap	Each	
	Normal			1 – 5
	Long			

Sizing charts

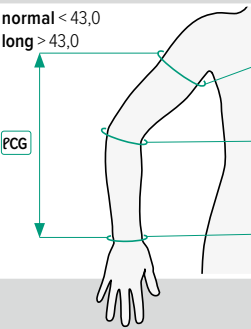
Compression Wrap for upper extremities

Adjustable compression system

Juzo Compression Wrap Hand

Art. 6000 UHR/UHL		Standard sizes · Circumferences in cm				
		1 / XS	2 / S	3 / M	4 / L	5 / XL
	cA	15.5 – 17.5	17.5 – 19.5	19.5 – 22.0	22.0 – 24.5	24.5 – 28.0
	cC	13.0 – 17.0	15.0 – 19.0	17.0 – 21.5	20.0 – 24.0	21.0 – 25.0

Juzo Compression Wrap Arm

Art. 6000 UA/UAG	Lengths in cm		Standard sizes · Circumferences in cm				
			1 / XS	2 / S	3 / M	4 / L	5 / XL
	normal < 43,0	cG	27.0 – 36.0	30.0 – 39.0	33.0 – 44.0	35.0 – 49.0	40.0 – 56.0
	long > 43,0	cE	23.0 – 32.0	24.0 – 34.0	26.0 – 36.0	29.0 – 42.0	31.0 – 46.0
		cC	16.0 – 19.0	17.0 – 20.0	18.0 – 22.0	20.0 – 26.0	21.0 – 29.0

Additional products

Compression Wrap

Adjustable compression system

Complementary additional products are available for Compression Wrap. They support patients in adapting their garments to individual requirements and are therefore perfectly suited for self-banding.



Compression Wrap Touch Fastener

Art. 6000F2

Double-sided Compression Wrap touch fasteners make it easy to put on the products and to adjust them in the event of fluctuations in body circumference.




Compression Wrap Extender

Art. 6000EXT

The Compression Wrap extender is used for the individual extension of a segment or pressure increase. It can be cut to size as required.

Product range

	Art.	Model	Unit
	6000F1	Touch fasteners	Set: 4 pieces
	Size: 40 mm	Foot wrap, hand wrap	•
	6000F2	Touch fasteners	Set: 4 pieces
	Size: 75 mm	Calf wrap	•
	6000F3	Touch fasteners	Set: 5 pieces
	Size: 85 mm	Calf wrap long, arm wrap	•
	6000F4	Touch fasteners	Set: 5 pieces
	Size: 95 mm	Knee wrap, thigh wrap, arm wrap long	•
	6000F5	Touch fasteners	Set: 3 pieces
	Size: 105 mm	Thigh wrap long	•
	6000F6	Touch fasteners	Set: 4 pieces
	Size: 50 mm	Foot wrap long	•
	6000EXT	Extender	Piece
	Size: 120 x 8.5 cm		•

Liner

Liner stockings for adjustable compression systems



Liner

Art. 6082

Liner stockings are liner stockings for use under any wrap system. They are gentle on sensitive skin and protect any dressing there may be while the compression system is put on and taken off. These black stockings come as a pair. Liner is available in two sizes each as calf-high or thigh-high stockings.

Colours



Black

Product range

	Art. 6082 Type	Model	Standard sizes
	AD	Below-knee stockings	
		Set with 2 stockings	1 – 2
	AG	Thigh-high stockings	
		Set with 2 stockings	1 – 2

Sizing chart

Juzo Liner

Art. 6082		Standard sizes · Circumferences in cm	
Lengths in cm		I / XS	II / S
	PG AG normal 80.0	50.0 – 75.0	76.0 – 99.0
	PD AD normal 45.0	29.0 – 48.0	48.0 – 69.0

INTRA

Liner stocking with SkinProtect® Silver

Adjustable compression system



INTRA Base layer below-knee stocking
Art. 0096, Art. 0086 (extraweit)

INTRA Base layer sock
Art. 0094

These soft-fabric stockings are especially comfortable to wear and are also suitable as a compression aid for patients with extremely sensitive skin. The stockings offer permanent skin protection during donning and doffing and, thanks to the knitted-in silver thread, also have an antibacterial effect. They are ideally suited for incorporating wound dressings or securing special pressure pads.

Colour



Silver



Product range

	Style	INTRA	Standard sizes
	AD	Base layer below-knee stocking SkinProtect® Silver	
	Art. 0096 Art. 0086	One-Size extra wide for calf circumference > 50 cm	• •
	AB	Base layer sock SkinProtect® Silver	
	Art. 0094	One-Size	•

Sizing chart

Art. 0096, Art. 0086 (extra wide)

series sizes · circumference in cm

INTRA Base layer below-knee stocking silver

< 50,0

extra wide
> 50,0

Additional information

Adjustable compression system

When do we recommend which product?

ACS Light

- compression up to 40 mmHg
- with secondary conditions that require low pressure (e.g. mild PAD, peripheral neuropathy)
- for patients who cannot tolerate high compression
- for anatomical features that require increased material elasticity (e.g. pronounced skin flaps in the area of the thigh)
- for a temporary increase in compression for flat knit products

Compression Wrap

- compression 30 – 60 mmHg
- if very high working pressure is required (e.g. in cases of venous leg ulcers)
- for pronounced fibrosis
- for very pronounced lymphoedema (e.g. primary lymphoedema, lymphoedema in stages II - III)



Contraindications

SoftCompress, ACS Light and Compression Wrap

Absolute contraindications

- advanced peripheral arterial occlusive disease (if one of these parameters applies ABPI < 0.5, ankle artery pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO₂ < 20 mmHg instep). When inelastic materials are used, compression garments can still be worn with an ankle artery pressure between 50 and 60 mmHg with continual clinical supervision.
- decompensated cardiac insufficiency (NYHA III + IV)
- septic phlebitis
- phlegmasia coerulea dolens
- acute bacterial, viral or allergic inflammation
- swelling of the feet or legs with unknown cause

Relative contraindications

- Pronounced weeping dermatoses
- Incompatibility with pressure or product ingredients
- Severe sensitivity disorders in the extremities
- Advanced peripheral neuropathy (e.g. in cases of diabetes mellitus)
- Primary chronic polyarthritis
- Peripheral artery disease (PAD) stage I/II
- Malignant lymphoedema

Important:

If there are open wounds in the area to be treated, the affected area has to be covered with appropriate dressing or bandaging after consulting with a doctor, before SoftCompress or ACS Light can be applied.

Complementary products for oedema therapy

Skin care



Derma Lotion

Art. 0950 (500 ml)

Art. 0925 (125 ml)

Moisturising lotion with 4 % urea for targeted skincare in cases of lymphoedema and lipoedema



Skin Protect

Art. 0757

Skin regeneration and acute care of stressed, sensitive areas of the skin, even in cases of venous disorders. The antimicrobial effect of silver reduces the risk of infection.



Anti-itch spray

Art. 0583 (150 ml)

The acute care against itching. The active ingredient polidocanol soothes the skin and can prevent itching for several hours.

Stocking gloves



Stocking gloves Super-Grip

Art. 0112

Extra fine textile glove with latex finish for gentle donning and better spreading of the knitted fabric

Size II (S)* = up to 8 cm handwidth

Size III (M)* = up to 9 cm handwidth

Size IV (L)* = up to 10 cm handwidth

Size V (XL)* = up to 11 cm handwidth



Stocking gloves Top-Grip

Art. 0108

Breathable, sensitive and durable textile gloves made of nylon with nitrile coating

Size II (S)* = up to 8 cm handwidth

Size III (M)* = up to 9 cm handwidth

Size IV (L)* = up to 10 cm handwidth

Size V (XL)* = up to 11 cm handwidth

* The size is determined by measuring the hand width at the finger baseline

Laundrying care



CLEAN detergent

Art. 0121 (250 ml)

Particularly mild and economically concentrated detergent for gentle cleaning of compression garments.



Laundry bag

Art. 1007

Protects your stockings and tights during machine wash quick and easy to fill by means of zipper.



Silver laundry bag

Art. 0891

With pure silver fabric (0% nano particles) to combat unpleasant smelling laundry, even at low temperatures.

Skin glue



Adhesive lotion

Art. 0126 (Roll-On 60 ml)

The skin-friendly, residue-free removable adhesive lotion gives stockings, supports and other textiles safe and permanent hold. After wearing the adhesive lotion is removable with water or with a moist cloth within seconds.

Complementary products for oedema therapy

Compression pads



Compression pad for Bisgaard area

Art. 0123 (LA/RI left outside / right inside)
Art. 0124 (RA/LI right outside / left inside)
Left and right of the achilles tendon.
Covered with textile material.



Compression pad for the hand – short

Art. 0111
For the back and palm of the hand. For treating oedema and for cone shaped parts of the hand.



Compression pad for the hand – long

Art. 0159
For the hand. Enables additional pressure. Wrist overlapping.



Compression pad silicone

Art. 0081 KL
Art. 0081 GR
naked (without cover)



Compression pad silicone

Art. 0082 KL
Art. 0082 GR
covered with elastic material

Positioning and supporting aid for arm and hand



Lymphatic cushion

Art. 0550 (Size I circumference forearm ≤ 30 cm)

Art. 0550 (Size II circumference forearm > 30 cm)

The treatment of lymphatic oedema and serious injuries of the arm and hand is supported by an elevated resting position. Swelling is reduced or even prevented. Improves back flow of the blood to the heart as well as the removal of the lymph fluid. The Lymphatic Cushion is a valuable treatment support, especially within the Complex Physical Decongestive Therapy (CPDT).

Protection for bandages



Double foot scraper coarse/fine, water-resistant

Art. 0956

Double-sided callus file for gentle foot care that is kind to the skin.



Shoehorn long appr 65 cm

Art. 0935

Stocking- and feet-friendly Shoehorn

SoftCompress

☐ Order ☐ Quotation only

Company stamp, contact person, phone (in block letters)		Patient details <input type="checkbox"/> Photo documentation will follow by e-mail ¹	
		Order no. ² :	
		Previous order no./Quotation no./Date:	
Customer no.:	Date:	Quantity:	<input type="checkbox"/> Piece <input type="checkbox"/> Pair <input type="checkbox"/> Female <input type="checkbox"/> Male <input type="checkbox"/> Diverse

☐ Left arm

☐ Right arm

Circumferences (c) in cm · Lengths (P) in cm

Diagram showing measurement points for the left (L) and right (R) arms. Circumference points are labeled cG, cF, cE, cD, cC', and cC. Length points are labeled P_{CG}, P_{CF}, P_{CE}, P_{CD}, P_{CC'}, and P_{CC}. A minimum of 6 cm is indicated between the wrist and the elbow.

Model Arm
☐ Bandage Arm (Art. 6822)

Border
(We recommend a diagonal top.)
☐ Diagonal top ☐ Straight top

☐ Left hand

☐ Right hand

Circumferences (c) in cm · Lengths (P) in cm

cA cA
P_{AC} P_{AC}

Diagram showing measurement points for the left (L) and right (R) hands. Circumference points are labeled cA and cC. Length points are labeled P_{AC}.

☐ Left leg

☐ Right leg

Circumferences (c) in cm · Lengths (P) in cm

Diagram showing measurement points for the left (L) and right (R) legs. Circumference points are labeled cG, cF, cE, cD, cC, cB', cB, cY, and cA. Length points are labeled P_G, P_F, P_E, P_D, P_C, P_{B'}, P_B, P_A, and P_Z. A minimum of 6 cm is indicated between the ankle and the knee.

Lengths in cm	
Left leg	Right leg
P _G :	P _G :
P _F :	P _F :
P _E :	P _E :
P _D :	P _D :
P _C :	P _C :
P _{B'} :	P _{B'} :
P _B :	P _B :

Ball stub/diagonal foot
(indicate "PA")
Interior: Interior:
Exterior: Exterior:

Foot length (back of heel to "cA")
..... P_A
Total foot length only if desired with closed toes
..... P_Z

Model Leg
☐ Bandage Lower Leg (Art. 6830)
☐ Bandage Lower Leg and Thigh (Art. 6820)

Border
(We recommend a diagonal top.)
☐ Diagonal top ☐ Straight top

Toe
☐ Open foot sole (standard)
☐ Closed foot sole
☐ Closed toes
☐ Open toes
☐ Diagonal foot
(Indicate internal and external foot length.)
☐ Straight foot

☐ Please send a new order form

Special requests (in block letters):
.....
.....
.....
.....
.....
.....
.....
.....

¹ Due to the principle of data minimisation under data protection law, we recommend that you only send in a photo in the case of difficult anatomical features.
² If the patient name is provided, the company placing the order confirms that it has obtained lawful consent in advance to forward and process the data of the affected patient.

SoftCompress

☐ Order ☐ Quotation only

Company stamp, contact person, phone (in block letters)		Patient details <input type="checkbox"/> Photo documentation will follow by e-mail¹	
		Order no. ² :	
		Previous order no. / Quotation no. / Date:	
Customer no.:	Date:	Quantity: <input type="checkbox"/> Piece <input type="checkbox"/> Pair	<input type="checkbox"/> Female <input type="checkbox"/> Male <input type="checkbox"/> Diverse

	Genital pad <input type="checkbox"/> Female (Art. 6839) <input type="checkbox"/> Male (Art. 6840) Quantity: Quantity:
	Measurements in cm H (belly / max. novel to crotch): H ¹ (bottom of pubic bone to perineum): B (between the hip joints): B ¹ (width crotch / scrotum):

	Breast Pad (Art. 6837) Quantity: Measurements in cm Cup: H (length entire chest along sternum): B (breast bone to shoulder blade):
--	---

	Breast bandage (Art. 6838) Quantity: Cup: Measurements in cm cB (breast circumference): cU (underbreast circumference):
--	--

	Pad <input type="checkbox"/> Elbow joint (Art. 6842) <input type="checkbox"/> Knee joint (Art. 6845) Quantity: Quantity: Measurements in cm
	Measurements in cm H (max. 16 cm): B (max. 13 cm):

		Hand Pad (Art. 6844) Quantity: Measurements in cm
		H ¹ : B¹: H ² : B²: Cut-out middle finger · H ³ : B³: Cut-out ring finger · H ⁴ : B⁴:

☐ Please send a new order form¹ Due to the principle of data minimisation under data protection law, we recommend that you only send in a photo in the case of difficult anatomical features.² If the patient name is provided, the company placing the order confirms that it has obtained lawful consent in advance to forward and process the data of the affected patient.

Thought of everything?

Checklist / Order form

Company Stamp, contact, phone (in block letters)

[illegible]

COMES

Smart Tape Measure

Fast, exact, no transmission errors

- Measured values can be sent via Bluetooth to computers/notebooks/tablets
- Undo function
- Flexible measuring tape with mm-scale
- Hook mechanism for quick circumference measurement
- Adjusting function for tightening/loosening
- Integrated battery with USB charging function
- No software installation required



Pharmacies and COMPRESSANA customers obtain Compression systems for decongestion, SoftCompress BandageAids and Adjustable compression systems (ACS) through COMPRESSANA GmbH.

www.entstauungssysteme.de

COMPRESSANA is a member of the Juzo Group.

The general terms and conditions of delivery and payment of COMPRESSANA GmbH apply.



COMPRESSANA GmbH
Böhmerwaldstr. 3
93073 Neutraubling
Germany
Phone: +49 (0) 9401 9226-0
E-Mail: info@compressana.de